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PREPARATION AND OPERATION

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Open the chair Squeeze together the two red levers under the seat to release the opening mechanism, gently drive the seat down till its complete opening.

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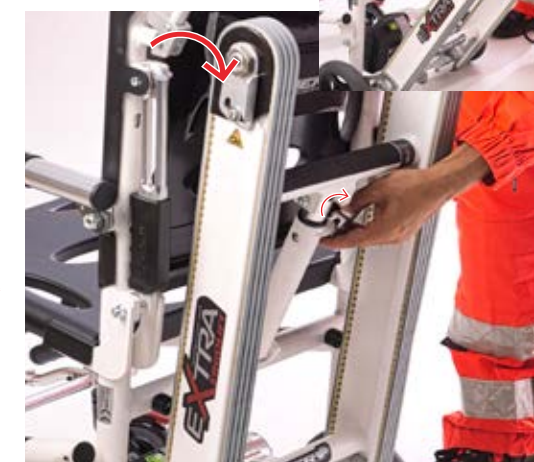
Open the tracks Pull the lever to the right and move the track frame outward till its full extension.

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Open the rear extendable handlebar Press down the lever positioned under the console and pull upward the handlebar.

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Insert the empty battery to the supplied battery charger. Check the main battery console charge status before and after each use.

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Setting up Insert the battery into the socket and firmly fix it by pressing it downward until when you hear a click.

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Turn on the electric engine. Switch on the green Power button located on the engine close to the battery.

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Turn on the console. Press any of the console buttons to turn it on. Press up or down the red levers to operate the tracks.

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Extend the frontal handles Turn the handles inward and pull off the bars till mid of full extension, then turn once more the handles outward to lock them in the chosen position.

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Squeeze together the two red levers under the seat to release the opening mechanism, lift the seat upward till its complete lock.

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Pull up the fixing system and drive the chair wheels on their location, press down the handle on top of the handlebar chair to completely fix it.

Fissaggi certificati 10G
Certified 10G fixing systems
Art. 975
per/for
Extra Ergolift
671 / 670br

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